tango reflex...

MULTI-MODALITY YAG/SLT LASER





Laser Floater Treatment Premium Capsulotomy Glaucoma Treatment

tango reflex.



EXCEPTIONAL VISUALIZATION

Featuring Ellex's proprietary Reflex Technology™, Tango Reflex's visualization in both on-axis and off-axis modes, combined with titratable illumination allows you to accurately visualize the posterior capsule as well as vitreous strands and opacities, and to assess position of floaters relative to the lens or retina.



TANGO REFLEX™ ENABLES
YOU TO **EXPAND THE SCOPE**OF CARE YOU'RE ABLE TO
PROVIDE, AND ALL AT THE

ABSOLUTELY HIGHEST STANDARDS OF SAFETY AND EFFICACY.

■ Reflex Technology[™] Treatment versatility, patient care quality


The effectiveness of laser vitreolysis and capsulotomy has been transformed through the development of Ellex's Reflex Technology™ platform, which includes True Coaxial Illumination (TCI™) that generates a titratable red reflex, a precise aiming beam and a superior energy beam profile - all within a unique slit lamp illumination tower design that converges and focuses your sight line, target illumination and treatment beam into one optical path.

■ True Coaxial Illumination (TCI™) Treat with greater accuracy

True Coaxial Illumination (TCI™) is at the heart of Tango Reflex™. It's technology that provides a stable and titratable red reflex across the entire working diameter within the pupil, that creates the highest degree of contrast, edge definition and detail shadowing for posterior capsule and throughout the vitreous.

LFT



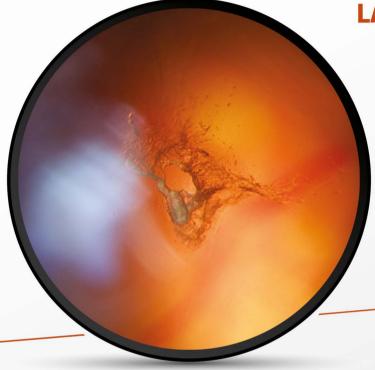

LASER VITREOLYSIS -TRANSFORM VISUAL ACUITY, IMPROVE PATIENTS' LIFE QUALITY

Image courtesy of Karl Brasse, MD

■ Reflex Technology™ A powerful solution to diagnose and manage symptomatic floaters

In the past, physicians have had to rely on vitrectomy as the only effective medical intervention for symptomatic floaters.

Due to the risk profile of the surgical procedure, patients have been told to live with symptomatic floaters. However, recent studies have shown that laser floater treatment known as laser vitreolysis is a viable option for the symptomatic relief of selected patients.¹

■ Tango Reflex™ Greater consistency in clinical results

Laser vitreolysis is a safe and effective first-line inoffice laser-based option to reduce or eliminate visual disturbance caused by symptomatic floaters.

Reflex Technology™ is a holistic laser solution for the treatment of debilitating floaters. Laser vitreolysis performed with Reflex Technology™ efficiently vaporizes hyaluronic acid and collagen strands to clear or reduce the size and/or amount of floaters in the vitreous.

Employing TCI™ to generate red reflex, Tango Reflex™ provides an enhanced visualization of the anterior and posterior vitreous. This minimizes the potential for focusing errors and the risk of damage to the natural lens or retina, providing a safe and highly effective tool for the treatment of vitreous strands and opacities.

HIDcap_m

PREMIUM CAPSULOTOMY

IT'S ACCURACY THAT
MINIMIZES FRINGES AND
TAGS AND PREVENTS LENS
DAMAGE - EVEN IF THE
LACK OF A RIDGE MAKES
THE CAPSULE ADHERE
TO THE OPTIC.



Image courtesy of Karl Brasse, MD

The benefits of Ellex's proprietary Reflex Technology™ extend beyond the safe treatment of floaters.

Employing TCI™ to visualize capsular fragments, Tango Reflex™ can be used to visualize and vaporize broken pieces of fragments and help prevent the common problem of sudden floater development after capsulotomy.

■ Green aiming beam

Tango Reflex™ features a precise green aiming beam that provides the highest degree of contrast in red reflex background, resulting in greater targeting accuracy which further enhances the safety profile of YAG laser treatments.

■ Precision in incision

Ellex's proprietary YAG laser cavity within Tango Reflex™ delivers a four nanosecond Ultra Gaussian pulse at high peak power and can typically achieve the industry's lowest optical breakdown energy at 1.5 mJ in air². That means you can perform capsulotomy at lower, more efficient energy levels. With less energy delivered into the eye, you'll be able to carry out HDcap™ capsulotomy with all types of IOLs, and with significantly less risk of lens pitting.

Choose Tango ReflexTM and you'll secure new levels of accuracy with $HDcap^{TM}$ capsulotomy - a perfectly centered, precise capsulotomy.



LIGHT STUDY IN FIGURES³



PATIENTS RANDOMLY ASSIGNED TO SLT (329 PATIENTS) OR EYE DROPS (323 PATIENTS).



74,2%

OF SLT PATIENTS REACHED TARGET IOP

AND WERE DROP-FREE AT 36 MONTHS.



5 TIMES LESS MEDICATION-DROP RELATED ADVERSE EVENTS* WITH SLT. *Aesthetic side effect or ocular reactions. Selective Laser Trabeculoplasty (SLT) is clinically proven by the LiGHT trial to be a safe and effective first-line therapy for the treatment of primary open angle glaucoma or ocular hypertension.³

Tango Reflex™ incorporates Ellex's proprietary SLT technology providing superior energy control, a sharp-edged aiming beam and the industry's fastest firing rate of four shots per second.⁴

With an enhanced view of the trabecular meshwork, you'll be able to perform SLT procedures faster and more accurately.

tango reflex...

TECHNICAL SPECIFICATIONS

SLT MODE

Q-switched, frequency doubled Nd:YAG Laser Source Wavelength

green: 532 nm

Energy 0.3 to 2.6 mJ per pulse, continuously variable

Pulse Width Spot Size 400 µm Illumination

LED light source **Aiming Beam** red 635 nm, adjustable intensity

Laser Source O-switched Nd:YAG Wavelength infrared: 1064 nm

Energy 0.3 to 10 ml per pulse, continuously variable

Pulse Width

Burst Mode 1, 2 and 3 pulses per burst, selectable Spot Size

8 µm

Offset (Anterior and Posterior) 0, 100 to 500 μm, continuously variable Illumination True Coaxial Illumination™ (Reflex Technology™) **Aiming Beam**

green 515 nm, adjustable intensity

COMMON FEATURES SPECIFICATION

Maximum allowable firing rate

Magnification

Cooling **Electrical Requirements**

Dimensions (HxWxD) Standard Accessories

Optional Accessories

4Hz (typical)2

optimized for enhanced anterior segment

visualization

Fan cooled

100–240 VAC, 50/60 Hz, 800 VA

31 kg, 68 lbs (laser only) 57 x 75 x 44 cm, 23 x 30 x 18 inches (laser only)

Total Solution™ tables, remote display, safety

glasses, laser safety sign, dust cover Tonometer mount, vitreolysis laser lens, SLT laser lens, capsulotomy and iridectomy laser lenses, footswitch, five-position magnification changer, beam splitter, "C" mount camera adapter, video camera

adapter, co-observation tube

Specifications are subject to change without notice

Tango Reflex™ has a CE Mark (Conformité Européenne) and is registered with US Food and Drug Administration (FDA) for the indications of Posterior Membranectomy, Capsulotomy, Laser Iridotomy and Selective Laser Trabeculoplasty (SLT).

Tango Reflex™ has a CE Mark (Conformité Européenne) for the indication of Laser Vitreolysis.

Laser Vitreolysis is not a cleared indication in US. Only intended for non-US based customers.

RIRI IOGRAPHY

- (1) Shah CP, Heier JS. YAG laser vitreolysis vs sham YAG vitreolysis for symptomatic vitreous floaters: A randomized clinical trial. JAMA Ophthalmol. 2017; 135(9): 918–923.10.1001/jamaophthalmol.2017.2388
 (2) based on system performance testing
 (3) Gazzard G, Konstantakopoulou E, Garway-Heath D, et al. Selective laser
- trabeculoplasty versus eye drops for first-line treatment of ocular hypertension Mar 9;393(10180):1505-16.
- (5) Katz LJ, Steinmann WC, Kabir A, Molineaux J, Wizov SS, Marcellino G; SLT/Med Study Group. Selective laser trabeculoplasty versus medical therapy as initial treatment of glaucoma; a prospective, randomized trial. J Glaucoma. 2012,21:460-8

www.ellex.com



LASER CLASS 2 Diode Laser 635mm. < Im/W Max CW & Diode Laser 515mm. < Im/W Max DW & Di

Headquarters

Quantel Medical Rue du Bois Joli - CS40015 63808 Cournon d'Auvergne - France Tel: +33 (0)4 73 745 745 Email: contact@quantelmedical.fr

Ellex Medical Pty Ltd 3-4 Second Avenue Mawson Lakes, SA 5095 Australia Tel: +61 (0)8 7074 8200

